


How to call the Scheme Manager

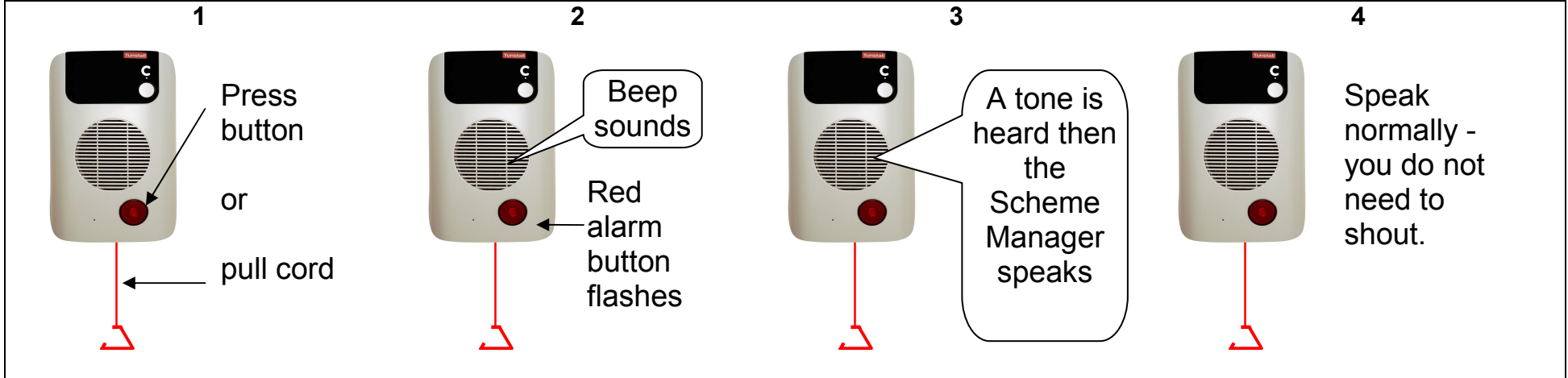

Amie Personal Trigger

Press to call Scheme Manager



Light will illuminate for 5 seconds

© 2008 Tunstall Group Ltd.

www.tunstall.co.uk

Privacy



If the button is pressed in the scheme manager can hear you. If the button is out then the scheme manager cannot hear you.

Privacy Override

If the scheme manager calls and the privacy button is out then the scheme manager cannot hear you. To let the scheme manager hear you just press the button on your Amie personal radio trigger once.
If you make an alarm call, by pressing a button or pulling a cord, the scheme manager will always be able to hear you.

Tunstall

Cancel at Source

1



In addition to the red alarm button and privacy button, your speech module has a Cancel at Source button. If this button is not pressed after an alarm call is raised the speech module will continue to raise alarm calls.

2



After an alarm has been raised, the scheme manager will visit you to press the cancel button. Alternatively, they may ask you to press the cancel button yourself.