

All the reassurance you need

Tunstall

TELECARE MANAGER QUICK REFERENCE CARD

1 To enter programming mode

- § Press the green c button until a tone sounds.
 - § Release the button
 - § The unit will say programming mode
 - § Press lit button 3 repeatedly to scroll through options
 - § The status button will flash while in programming mode.
-

2 To exit programming mode

- § Press the green c button
 - § Tones will confirm exit
 - § The screen will return to its normal status
 - § The status button will stop flashing
-

3 To assign a trigger

- § While in programming mode select "Assign trigger" option by pressing lit button 8 when assign trigger text is displayed.
- § Enter speech module number using the keypad
- § Press OK to confirm or keypad c button to backspace
- § Activate trigger
- § If required use button 3 to change location by scrolling through choices
- § Press lit button 8 to confirm or 4 to start again

All the reassurance you need

Tunstall

TELECARE MANAGER QUICK REFERENCE CARD

4 To delete a trigger

- § While in programming mode select "Delete trigger" option using lit button 8
 - § Activate the trigger
 - § The trigger details will be displayed
 - § Confirm deletion using button 8
-

5 To remove a dwelling

- § While in programming mode select "Remove Dwelling" option by pressing lit button 8.
 - § Enter dwelling number by using keypad
 - § Confirm number by pressing button 8
 - § The unit will display 'please wait searching'
 - § By pressing button 8 all triggers assigned to that dwelling will be deleted.
 - § Press button 4 to cancel
-

6 Status messages

- § The status button flashes
- § A tone will sound repeatedly
- § The screen will read "New system event"
- § Press Button 8 "View Event"
- § Press lit button 8 to accept the message

Telephone 01977-661234
E-mail sales@tunstall.co.uk
www.tunstallgroup.com

Part Number: D3607109A

Copyright © Tunstall Telecom Ltd 2003