

# CommunicaII Digital: Resident Quick Reference Guide

To raise an alarm, press the illuminated **Alarm Button (6)** once, or press your personal pendant. Your room unit will inform you the alarm has been raised.

## 1. Privacy Buttons

Toggles between:

- Home / Away
- Care calls / No Care calls
- Door calls / No Door calls

## 2. Call Button (optional)

Opens second screen with keypad to dial other residents

## 3. Microphone

## 4. Home Button

Two functions:

- Turns on display
- Returns to home screen

## 5. Settings

Opens second screen to view and control privacy status

## 8. Speaker

## 7. Camera (optional)

For resident-to-resident video calling

## 6. Alarm Button



Press once to call for help





# Communicall Digital: Resident Quick Reference Guide

## Video Door Calls


Video door calls will be presented on screen as per this image.

- To answer, press 
- To decline the call, press 

Once answered, the green telephone button will disappear. You can now:

- End the call **not allowing entry** by pressing 
- End the call and **allow entry** by pressing 



Pressing the  button at any time will present descriptions beside all buttons to confirm their function

## "I'm OK" (*optional*)

When a scheduled "I'm OK" prompt/button appears on screen, simply press the button to confirm you are ok. Help will be notified if you do **not** press the button within the scheduled times.

Please press here to confirm you are ok.

