

Product Update

Vibby Firmware Upgrade

We would like to inform you of some recent functionality changes that have been made to the Vibby fall detector (67605/22A1).

The changes to the product include:

- Position of the Green LED changed for better view (closer to the red LED)
- The SOS push button has a longer vibrator pattern from previous generation
- An update to the fall detection algorithm which means it is more sensitive to falls
- When putting the Vibby into storage mode the unit will continuously vibrate when pressing the button 5 times

There are no physical changes to the product.

These changes **do not affect** Vibby fall detectors with the part number 67605/20B1 which will continue to work as expected.

In order to identify which Vibby fall detectors you may have in stock, please see the two images below.



Old Model 67605/20B1



New Model 67605/22A1

On the back of the Vibby, the label shows the new part number and issue (shown above).

The Vibby installation guide has been updated to reflect these changes, the new version can be found at tunstall.co.uk/vibby

For more information, please contact the Helpdesk on **01977 660204** or email helpdesk@tunstall.com